



"It has increased my confidence
and helped 100%. I do not have
any more pain."

Lisa

AGE: 56
MONALISA TOUCH
PATIENT



A Treatment for Vaginal and Vulvar Health.

Renewing intimacy and changing lives.

MonaLisa Touch



MonaLisa Touch® is the only technology for vaginal and vulvar health with:

- 30+ published clinical studies
- Ongoing studies with top academic institutions
- Embraced by hundreds of experts in female pelvic health



Lisa

AGE: 34
MONALISA TOUCH
PATIENT

"MonaLisa Touch is one of the most significant advancements in female pelvic health in decades. It is proving to be life changing for patients and game changing for clinicians. It is among the most rewarding treatments we are privileged to offer our patients."

Jeffrey Dell MD,
FACOG, FACS

"The MonaLisa Touch has finally given us an effective treatment for a condition that millions of women have been told "there is nothing else that can be done". So far the women I have treated have been amazed at their results."

Vickie Lovin, MD

MonaLisa Touch helps restore vaginal health by generating new collagen, elastin and vascularization.


3 treatments of < 5 minutes each*

In-office procedure

Minimal side effects

No downtime


Tens of thousands of women successfully treated since 2012

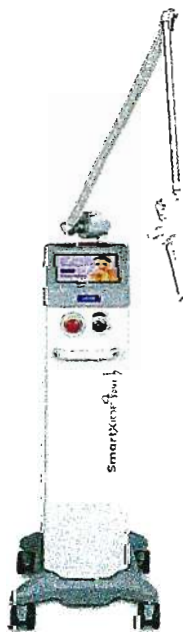

MonaLisa Touch®

* A maintenance treatment is recommended at ~ 1 year or as determined by the physician



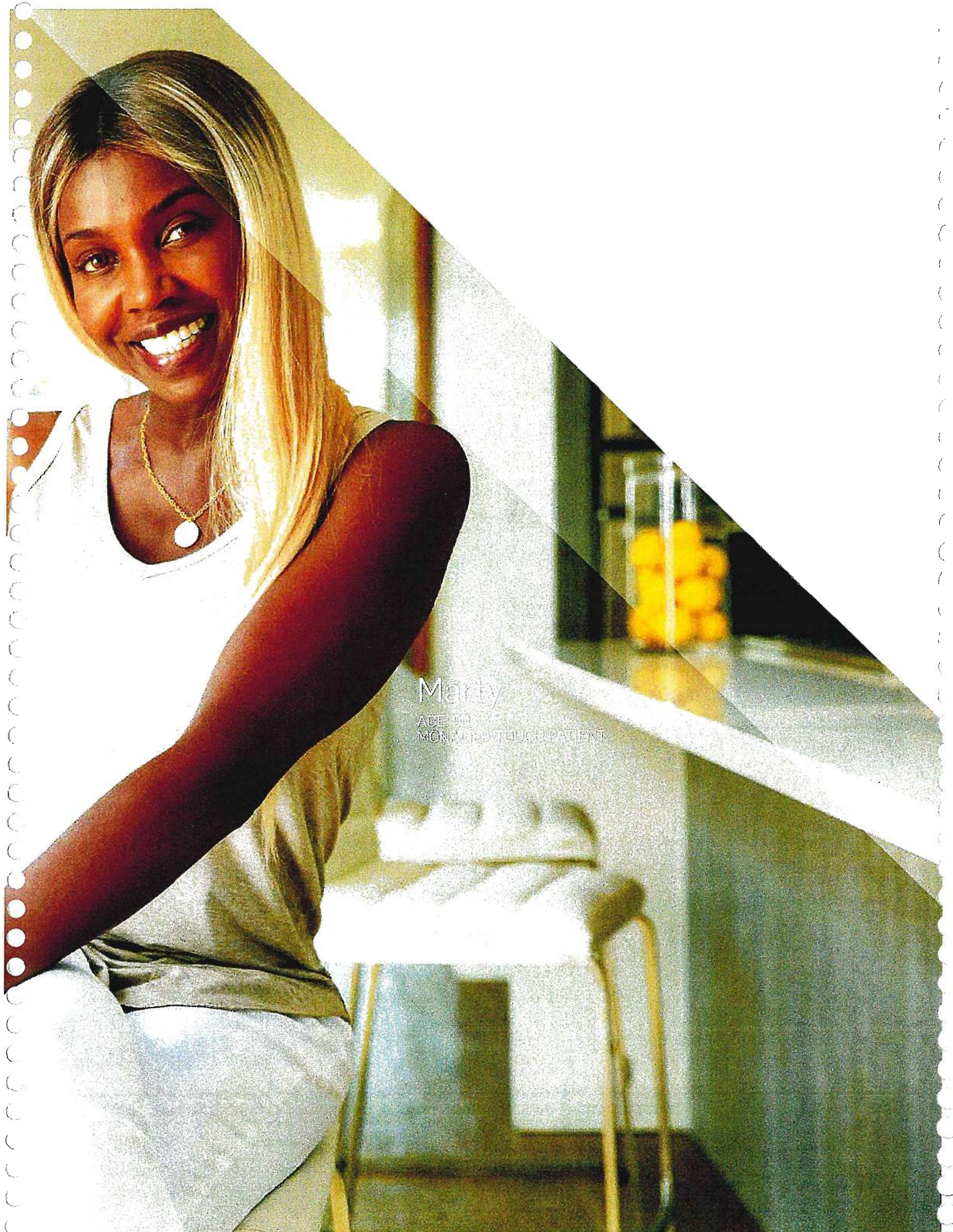

Marty




AGE 50
MONTHS TOUCH PATIENT

Clinically Proven

- Delivers fractional CO₂ laser energy to vaginal mucosa and vulvar tissue
- Ablates and coagulates tissue to promote neocollagenesis and angiogenesis
- 30+ published and ongoing studies

Stacking Technology

- Proprietary software enables repetitive firing in same microspot
- Stacking achieves deeper penetration into lamina propria
- Smart Stack 3 allows energy to penetrate beyond moistened epithelium into lamina propria for 2nd and 3rd treatments

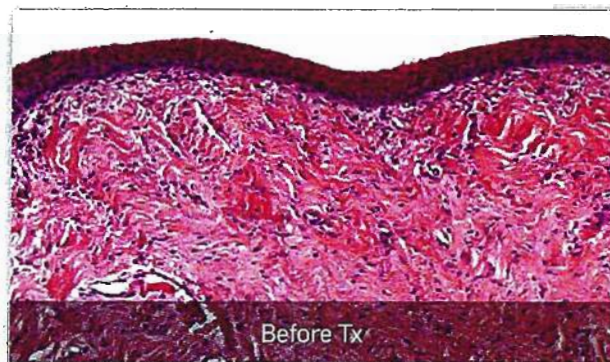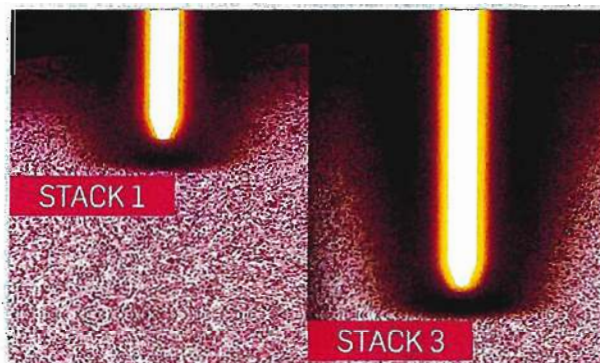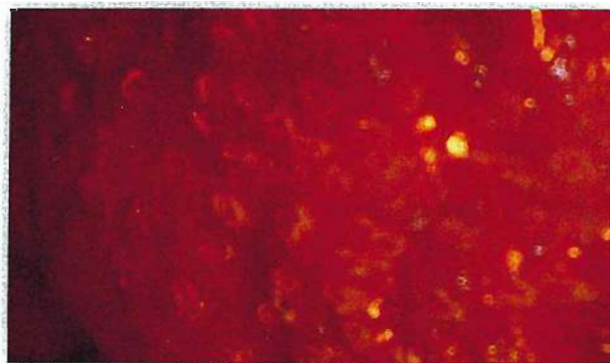

Before

Vaginal mucosa in the basal condition with a thinner epithelium



After

Follow-up was at 2 months—2 weeks after 2nd treatment with the MonaLisa Touch

New collagen, elastin and vascularization.

Histology courtesy of Prof. A. Calligaris
University of Pavia, Italy.

The Proof is in the Histology.

In this patient histology, the thicker epithelium and shedding of numerous big cells from the free surface, together with the larger diameter of epithelial cells rich in glycogen, demonstrate the restored metabolic trophism and dynamics of the whole epithelium.

For more information visit smilemonalisa.com/physician

MonaLisa Touch[®]



SmartXide Touch System	
Model	C60
Type of Laser Source	CO ₂ - RF Excited
Wavelength	10,600 nm
Beam Mode	TEM ₀₀
Power	60W max
Delivery System	7 mirror articulated arm
Aiming Beam	Diode laser @ 635 nm-4 mW Intensity 2%-100% Possibility to switch off when lasing (DOWL)
Control Panel	Touch Screen
Electrical Requirements	100 to 230 Vac - 50/60 Hz - 1,200 VA
Size (H x W x D)	46" X 16" X 21" (118cm X 42cm X 54cm)
Weight	137 lbs. (62kg)

HiScan DOT Scanning Unit	
Dwell Time	100-2,000 μ s
Spacing	0-2,000 μ m
Scan Mode	Normal, Interlaced, SmartTrack
Emission Mode	SP - DP - HP pulse
Smart Stack	1-5
Shape	Square
Probes	90° internal vaginal probe (x3), 23 mm Vaginal ring for 90° internal vaginal probe (x3) External handpiece (x3) 90° internal/narrow/vaginal probe, 16 mm (Optional)



INTERNAL VAGINAL PROBE AND RING



EXTERNAL HANDPIECE



INTERNAL NARROW VAGINAL PROBE

CYNOSURE
A Hologic Company

DEKA
The Code of Excellence

5 Carlisle Road
Westford, Massachusetts 01886 USA
TELEPHONE 978 256 4200
TOLL FREE 800 886 2966
www.cynosure.com

SOURCES 1 US Census, 2010, 2 Menopause, 2013:20(9):888-902, 3 Menopause, 2011:18(11):1160-1171

Physicians and patients were compensated for their product endorsement.
MonaLisa Touch is a registered trademark of DEKA M.E.L.A. Srl - Calenzano - Italy.
Cynosure is a registered trademark of Hologic, Inc. ©2017 Hologic, Inc.
921-7031-001 - Rev 3 10/17

